

091206T4PTT

PERIOPERATIVE THEATRE TECHNOLOGY LEVEL 6

HE/OS/TT/CR/03/6/A

Maintain Perioperative Theatre Instruments and Apparatus

July/Aug 2023



**TVET CURRICULUM DEVELOPMENT, ASSESSMENT AND CERTIFICATION
COUNCIL (TVET CDACC)**

PRACTICAL ASSESSMENT

INSTRUCTIONS TO ASSESSOR

1. You are required to mark as the candidate performs the task.
2. You are required to take video clips at critical points
3. Ensure the candidate has a name tag and registration code at the back and front
4. Allocate 5 minutes to the candidate for preparation and 30 minutes to carry out the task

This paper consist of FOUR printed pages

**Assessor should check the question paper to ascertain that all pages are
printed as indicated and that no questions are missing.**

OBSERVATION CHECKLIST

Candidate's name and Reg. No.			
Assessor's name and Reg. code			
Unit(s) of Competency	Maintain Perioperative Theatre Instruments and Apparatus		
Venue of Assessment			
Date of assessment			
<i>(Indicate the marks available and marks obtained respectively.</i>			
<i>Award marks appropriately as guided for in the items for evaluation indicated. Give a brief comment where necessary)</i>			
Items to be evaluated:	Marks available	Marks obtained	Comments
1. Donned personal protective gears as per hospital protocol <ul style="list-style-type: none"> • Gloves • Goggles • Gown • Masks • Gumboots <i>(Award 1 mark for each or a zero)</i>	5		
2. Identified at least five instruments correctly with the right names <i>(Award 1 mark for each or a zero)</i>	5		
3. Identified at least five instruments with the correct function <i>(Award 1 mark for each or a zero)</i>	5		
4. Sorted instruments and classified them by function: <ul style="list-style-type: none"> • Cutting • Clamping 	5		

<ul style="list-style-type: none"> • Grasping • Suctioning • Retracting <p><i>(Award 1 mark for every correct classification or zero)</i></p>			
5. Checked instruments for functionality <i>(Award 2 marks or zero)</i>	2		
6. Packed instruments using an appropriate technique: <ul style="list-style-type: none"> • Packed instruments in a large tray on top of the green towel • Arranged heavy instruments at the bottom. • Used clips to group together forceps, scissors and towel clips. • Packed the rest of the instruments in the tray. <p><i>(Award 1 marks or zero for each)</i></p>	4		
7. Wrapped the instruments using the appropriate technique <i>(Award 2 marks or a zero)</i>	2		
8. Labelled the pack appropriately <ul style="list-style-type: none"> • Type of pack. • Date of sterilization, • Expiry date, • Name <p><i>(Award 1 mark for or a zero)</i></p>	4		
9. Disposed waste appropriately <i>(Award 2 marks or a zero)</i>	2		
10. Documented appropriately.	2		

